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4. Process the place of accession of the wash basin to a wall by means of sealant to prevent hit of water between the wash basin and a wall.

5. Connect the wash basin to the sewer network by means of a special siphon (3-pict. 11). This model of a surgical wash basin is made with an opening for the mixer (with a diameter of 34 + - 2 mm) or without opening for the mixer .

Arms, fastenings, the mixer, siphon aren't included in the package!

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**Additional recommendations:**

When installing the product we recommend that you contact a qualified technicians

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**Instructions for product care.**

The surface of the product is resistant to solvents, acids and alkalis contained in cleaning agents household goods. Use ordinary household detergents. Do not use harsh cleaners, such as metal mesh, and the like sponges. If, however, the surface is contaminated and it can not be cleaned with normal detergents, use the special detergents. Do not leave the detergent on the surface of the product, and then rinse it with water. When using detergent follow all instructions of the manufacturer.

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Durability - 20 years under normal storage, transportation , installation and operation.

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Product Warranty - 5 years, subject to the rules of storage, transportation, installation and operation.

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